


About the Dynavox Eyemax Device Plate (DP-DV2)

The Dynavox Eyemax Device Plate (DP-DV2) attaches a Mount'n Mover Quick Release Plate (or Universal Rotator) to a Dynavox Eyemax.

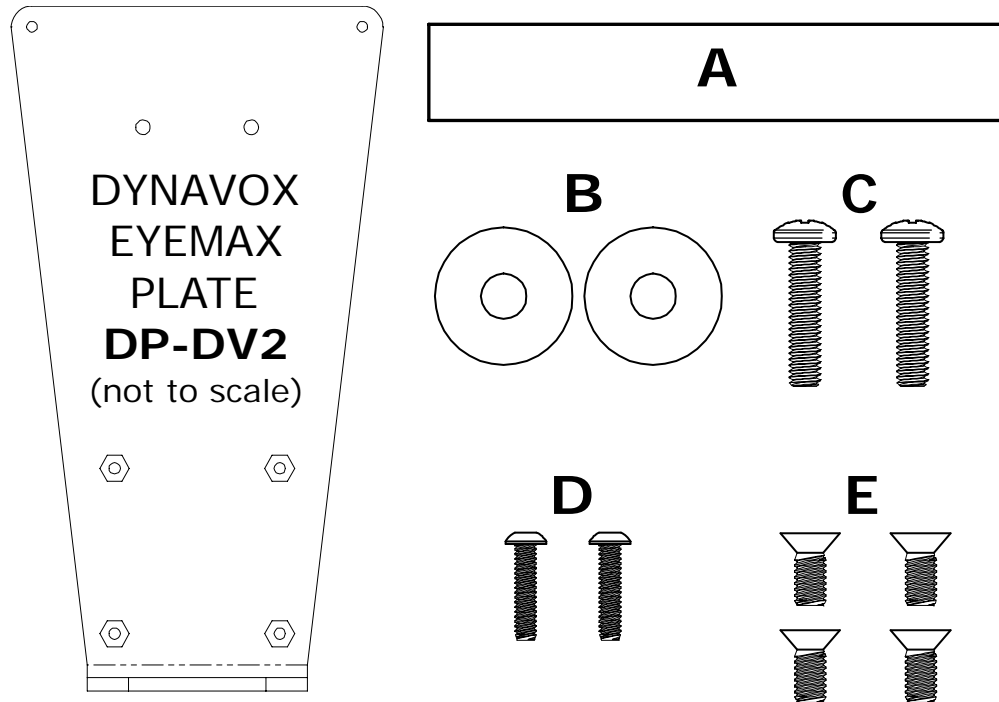

DP-DV2 Kit Contents:

1. Dynavox Eyemax Plate
2. Hardware:
 - A. (1x) - **Adhesive-backed cushion pad**
For protecting the finish of the Dynavox Eyemax device
 - B. (2x) - **Spacers**
For flush mounting of the DP-DV2 Plate
 - C. (2x) - **#8-32 x 3/4" Phillips Pan Head Screw**
For attaching the DP-DV2 plate onto the Dynavox Eyemax device
 - D. (2x) - **#4-40 x 1/2" Button Head Cap Screw**
For attaching the DP-DV2 plate to the Dynavox Eyemax device
 - E. (4x) - **#8-32 x 3/8" Phillips Flat Head Screw**
For attaching the Quick Release Plate (QRP) to the DP-DV2

Requirements:

Time : Approx 10 minutes

Tools :

- Phillips Screwdriver
(#2 is recommended)
- 3/32" Allen Key
for handle removal
- 1/16" Allen Key (**provided**)
for installing the DP-DV2



2637 27th Ave. S. Suite 209 • Minneapolis, MN 55406
 Ph: 612-724-7002 • Toll Free: 888-724-7002 • Fax: 888-311-4435
 info@blueskydesigns.us • www.mountnmover.com

Attaching the DP-DV2 to the Dynavox Eyemax



Prepare the Dynavox Eyemax for attachment.

Find a flat, clean surface and lay the Dynavox Eyemax screen side down.




Step 1/6 :

Attach the Adhesive-backed cushion pad (A) to the Dynavox Eyemax. Position the the pad directly under the rear label, centered on the device. Remove the liner and adhere.




Step 2/6 :

Using the 3/32" Hex key tool, remove the carrying handle. Set the handle safely aside (we will use it later).

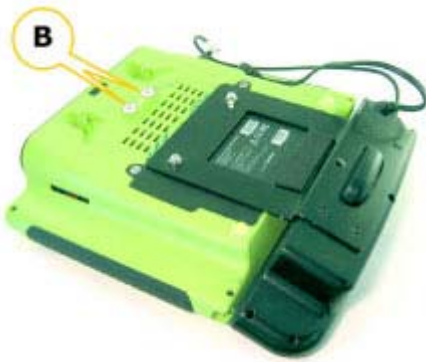


Step 3/6 :

Place the spacers **(B)** over the Dynavox Eyemax attachment holes.
Make sure that the spacers are roughly aligned with the threaded holes in the Dynavox Eyemax

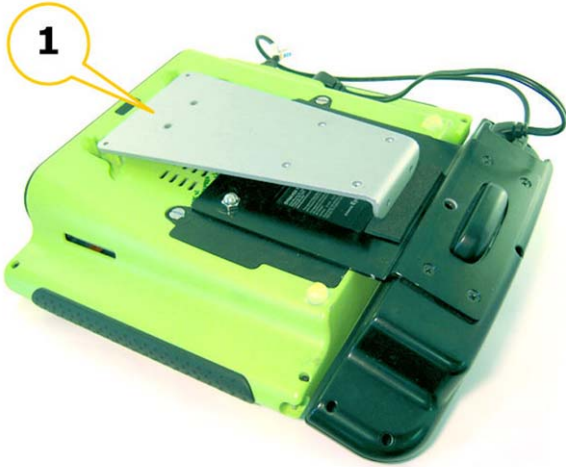

Step 4/6 :


Carefully place the Dynavox Eyemax Plate **(1)** on top of the spacers. Be sure to align the holes of the plate, spacers and the device.

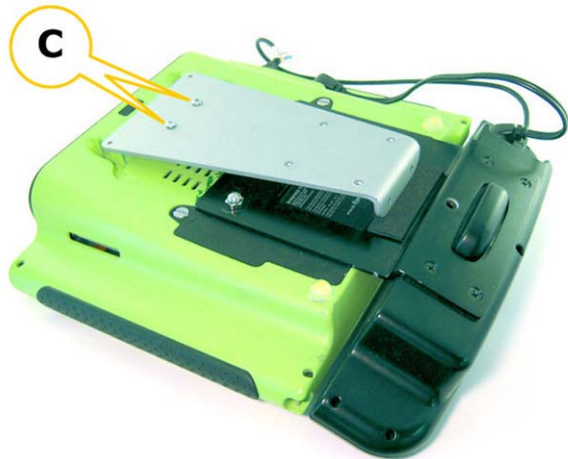

Step 5/6 :

Using the Phillips screwdriver, attach the DP-DV2 plate to the Dynavox Eyemax using **two** #8-32 x 3/4" Phillips Pan Head Screws **(C)**.


Make sure that the spacers are captured by the screws.

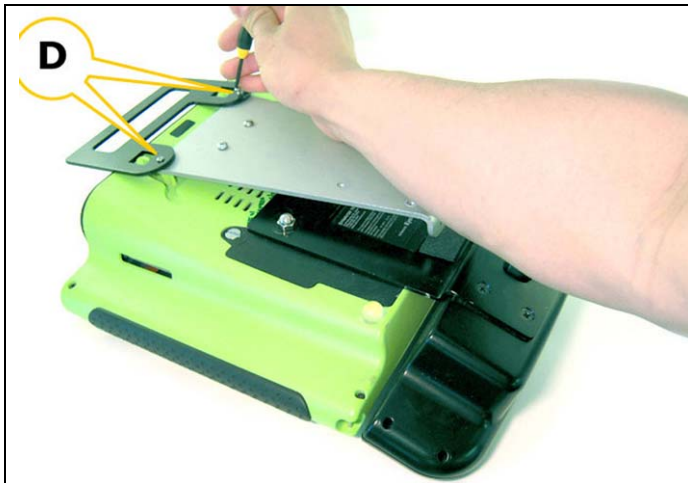

Step 6/6 :

Using a 1/16" Hex Key (*one is provided with each Mount'n Mover system*),
*Sandwich the DP-DV2 plate with the Dynavox Eyemax handle (removed in step 2).
Screw them together using **two** #4-40 x 1/2" Phillips Pan Head Screws **(D)**.

** re-attaching the handle is optional, if you decide the handle is unnecessary – you can omit the handle and screw down as normal.*

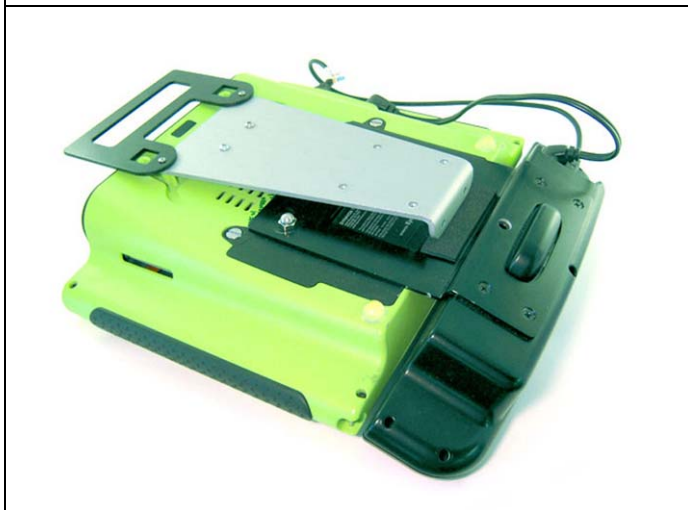


Congratulations!

The DP-DV2 is now attached to the Dynavox Device.


Please ensure that the fasteners are screwed down properly and the spacers are in place.

The Dynavox Eyemax is now ready to be mounted to a Quick Release Plate **(QRP)** or a Universal Rotator **(DP-UR)**

Attaching the Quick Release Plate (QRP) to the DP-DV2

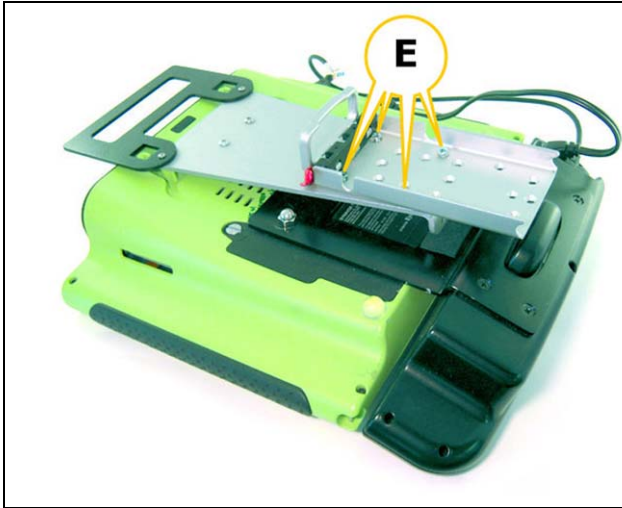

Using the Phillips screwdriver, attach the QRP to the DP-DV2 using **four** #8-32 x 3/8" Phillips Flat Head Screws (**E**). Make sure that the heads of these screws do not protrude from the QRP.

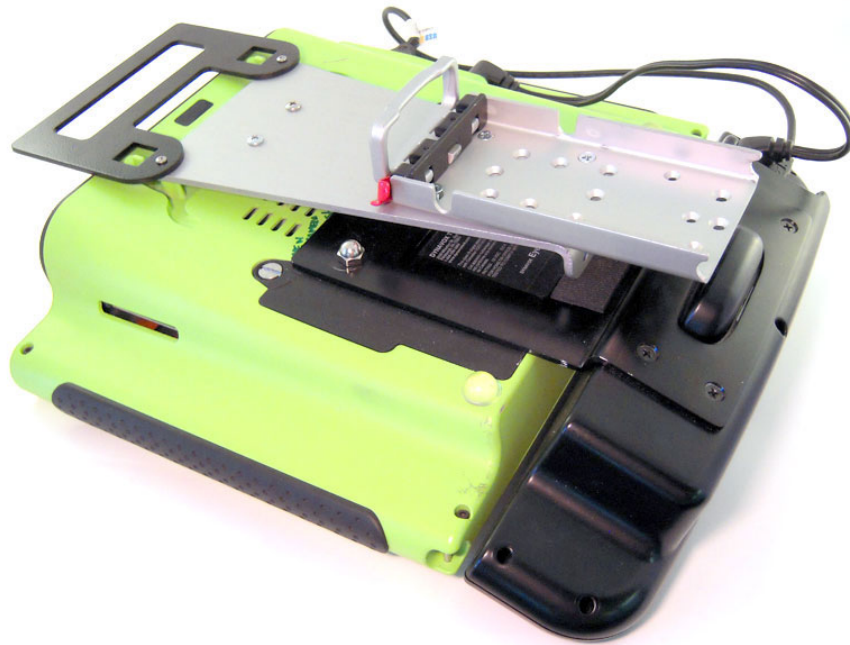

You are now ready to attach the Quick Release Plate to your Mount'n Mover! Refer to page 13 of your Mount'n Mover manual for instructions.
([you can also view the videos for it here](#))