


Adapter Plate 5 (WC-AP5) Multi-Use Plate

Scale 1 : 1

1/4-28 threaded Hole (Qty = 4)

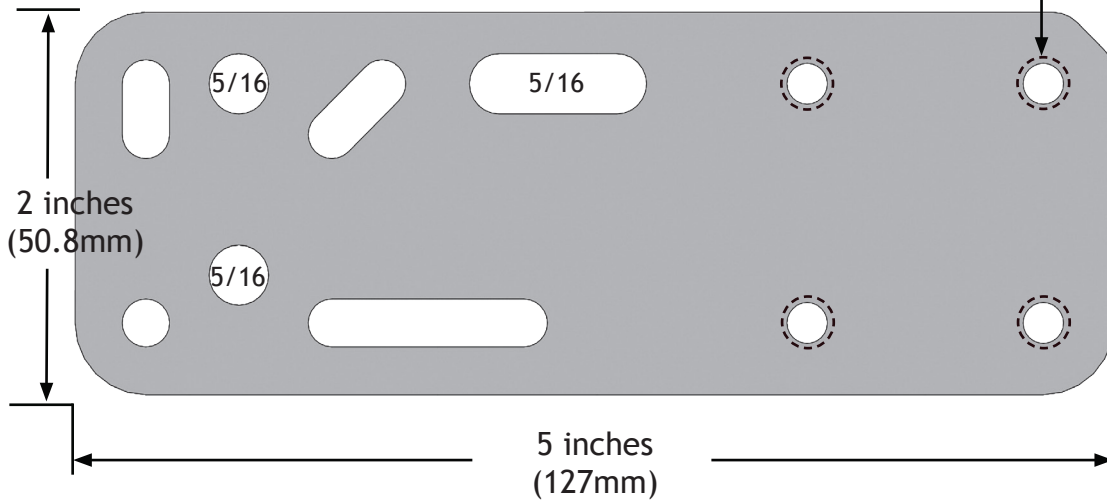

Set Includes:

(1) Adapter Plate 5 (WC-AP5)

Hardware - Plate to Plate Attachment

(2) 1/4-28 x 1/2" Socket head cap screws

Hardware - Through Holes in Frame:

(2) 1/4-28 x 1 5/8" Socket head cap screws

(2) 1/4-28 lock nut

Hardware - Unitrack:

(4) M6-1 x 14 Socket head cap screws

(4) M6-1 x 10 Square nut

WARNING!

Never attach two plates via threaded hole to threaded hole! This damages the threads and the connection can loosen!


Plate to Plate Attachment Options

- Non-threaded hole to threaded hole = Good
- Non-threaded hole to non-threaded hole with nut = Good
- **Threaded hole to threaded hole = Not good (will damage threads!)**

Hole Pairing Options

- Attaching with two holes = **BEST**
- Hole and slot (horizontal or vertical) = Good
- Slot and slot (both horizontal) = Good
- Two vertical slots = **NOT GOOD!** (could slip!)

* See Quick Guide and website for specifics

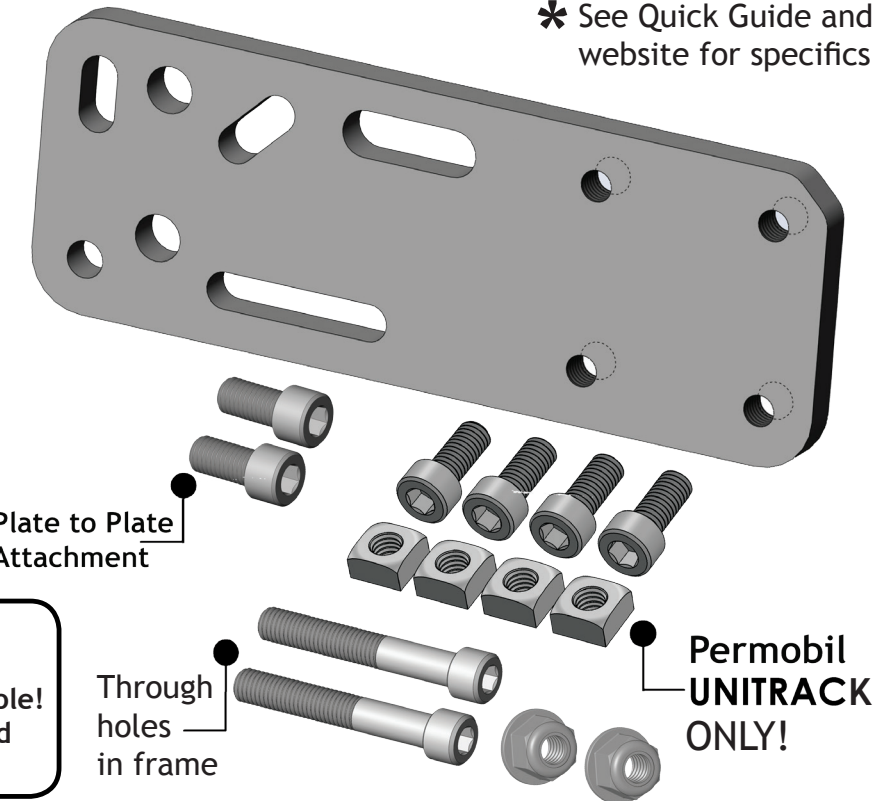

Mount'n Mover™
by BlueSky Designs

mountnmover.com

Product: WC-AP5, Adapter Plate 5

Weight: 10.6 oz. (0.3 kg)

1-888-724-7002 (toll free) or 1-612-724-7002

pg 1 of 2

Wheelchair Attachment Overview

* All plates are considered 'multi-use plates' and can be configured, combined, and used in a variety of ways.

