

## Cleaning and Disinfecting Your Mount'n Mover

BlueSky Designs encourages customers to help fight against the spread of germs by cleaning and disinfecting your BlueSky Designs products by following the steps below.

If you have a mobile device (communication aid, phone, tablet, iPad) attached to your mount, please refer to the manufacturers guidelines for cleaning and disinfecting. Always be sure to wear the appropriate personal protective equipment such as protective gloves and safety glasses.



### Cleaning and Disinfecting Your Mount, Hardware or Tray

1. Remove any large chunks, visible heavy dirt or debris that is attached to the mount and hardware.
2. Wipe it down with warm water and mild soap.
3. Use a cotton swab soaked with cleaner or an old toothbrush to get into any tight areas that were not accessible with the cleaning towel.
4. Let the device completely dry.

**USE**



**DON'T USE**

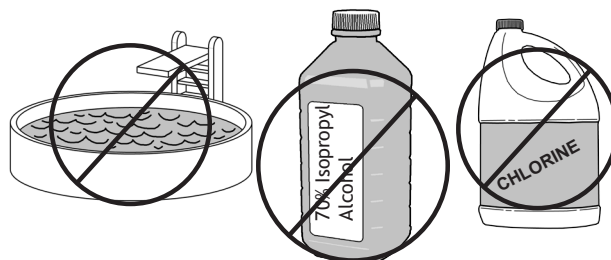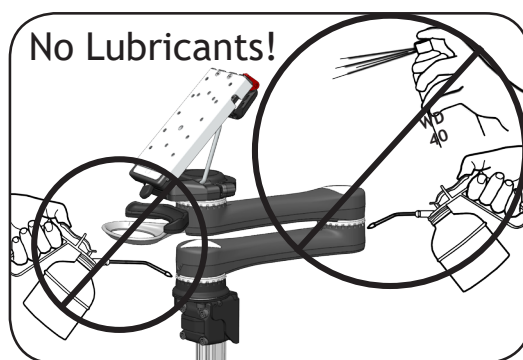

## Cleaning Polycarbonate Trays

Here's a sheet and a chart that focuses on what to use/not use on polycarbonate (our trays):  
<http://www.polycarbonatestore.com/how-to-clean-polycarbonate-panels/>

Cleaning Agents that have been found to be **COMPATIBLE** with polycarbonate sheets under laboratory conditions:

- Freon T.F.
- Joy
- Palmolive Liquid
- Top Job
- VM&P grade Naptha
- Brillilanize

Cleaning Agents that have been found to be **INCOMPATIBLE** with polycarbonate sheets under laboratory conditions and should **NOT** used:

- Lysol
  - Pinesol
  - Butyl Cellosolve
  - Isopropanol
  - Formula 409
- DO NOT USE**

If you have any questions about cleaning and disinfecting your BlueSky Design products, please contact us at [info@blueskydesigns.us](mailto:info@blueskydesigns.us) or 888-724-7002.