


Rifton Plate Attachment Instructions (WC-AP8X)

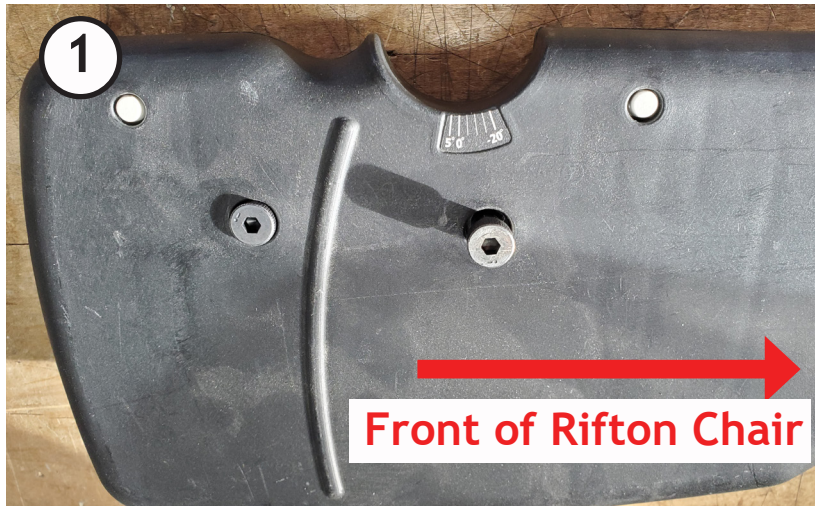


Remove one shoulder bolt from Rifton side panel. Use a **Torx 25 wrench** to do this. Do not remove both at once or the side panel may fall off.



Attach WC-AP8X adapter plate using longer shoulder bolts (included). Once one longer bolt is in, then remove the other Rifton bolt and attach the WC-AP8X with two longer shoulder bolts. Use a **4mm allen wrench** to tighten.



WC-AP8X attached to the Rifton side panel with two longer shoulder bolts. Note that the 4 threaded holes in the square pattern are towards the **front** of chair.

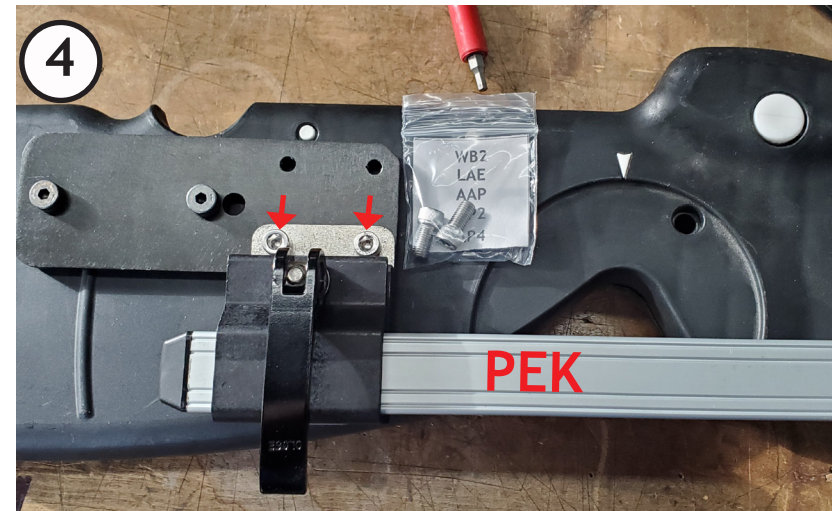

Attach the PEK (post extension kit) to the threaded holes on the WC-AP8X using two socket head 1/2 inch bolts (included).