

# Tobii Dynavox SC Tablet Attachment Instructions

( Uses regular ipad )

1




Place 3/8" spacers over threaded inserts in back of device.

2




Place the DP-QCP plate over the spacers and use the two M4-16 screws to attach it to the back of the device.

3

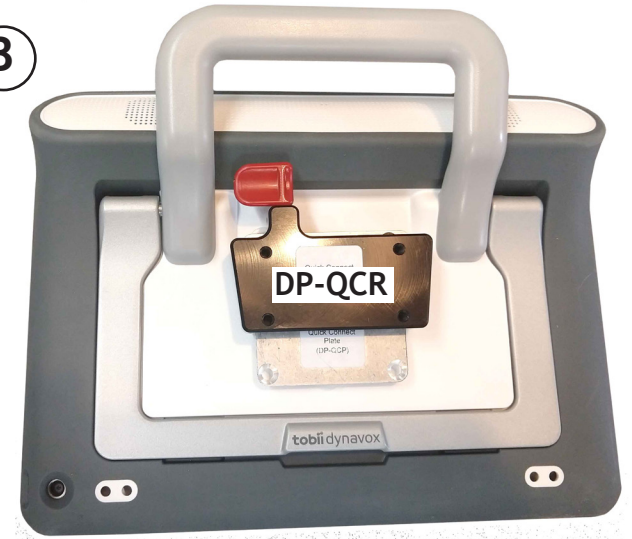


Slide the DP-QCR onto the DP-QCP until it clicks in place.

4

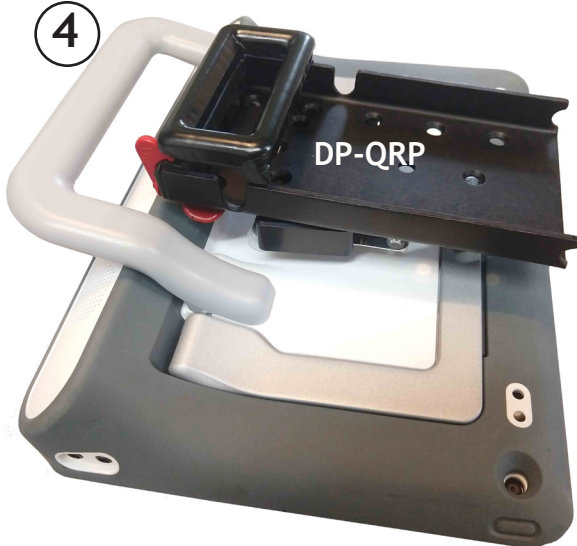

Attach the QRP onto the DP-QCR with the four screws included with the DP-QCR ( HS-Screw-X {QRP to DP-QCR} )

5

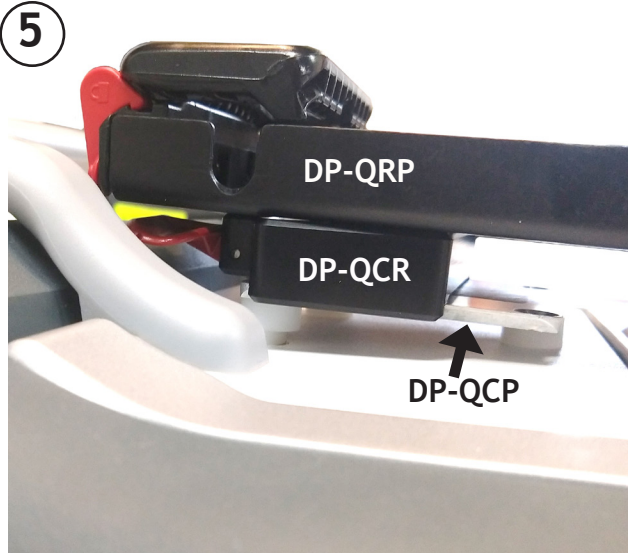

Close up of QRP attached to the DP-QCR attached to the DP-QCP.

## KEY



**DP-QCP**  
( Quick Connect Plate )



**DP-QCR**  
( Quick Connect Receiver )



**DP-QRP**  
( Quick Release Plate )

