



Attaches to: All Mount'n Mover device plates

Universal rotator to
device plate

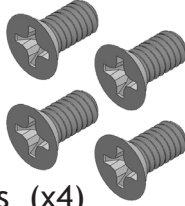

8-32 x 3/16 Flathead Screws (x4)



Loosen knob, rotate plate
45 degrees to expose holes
for attachment to
device plate

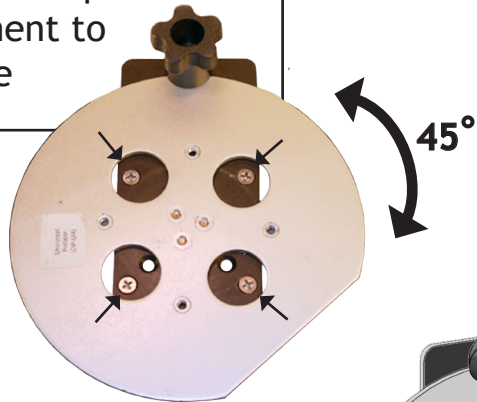

Align with holes to
attach Quick Release
Plate (QRP)



8-32 x 1/4" (x4)

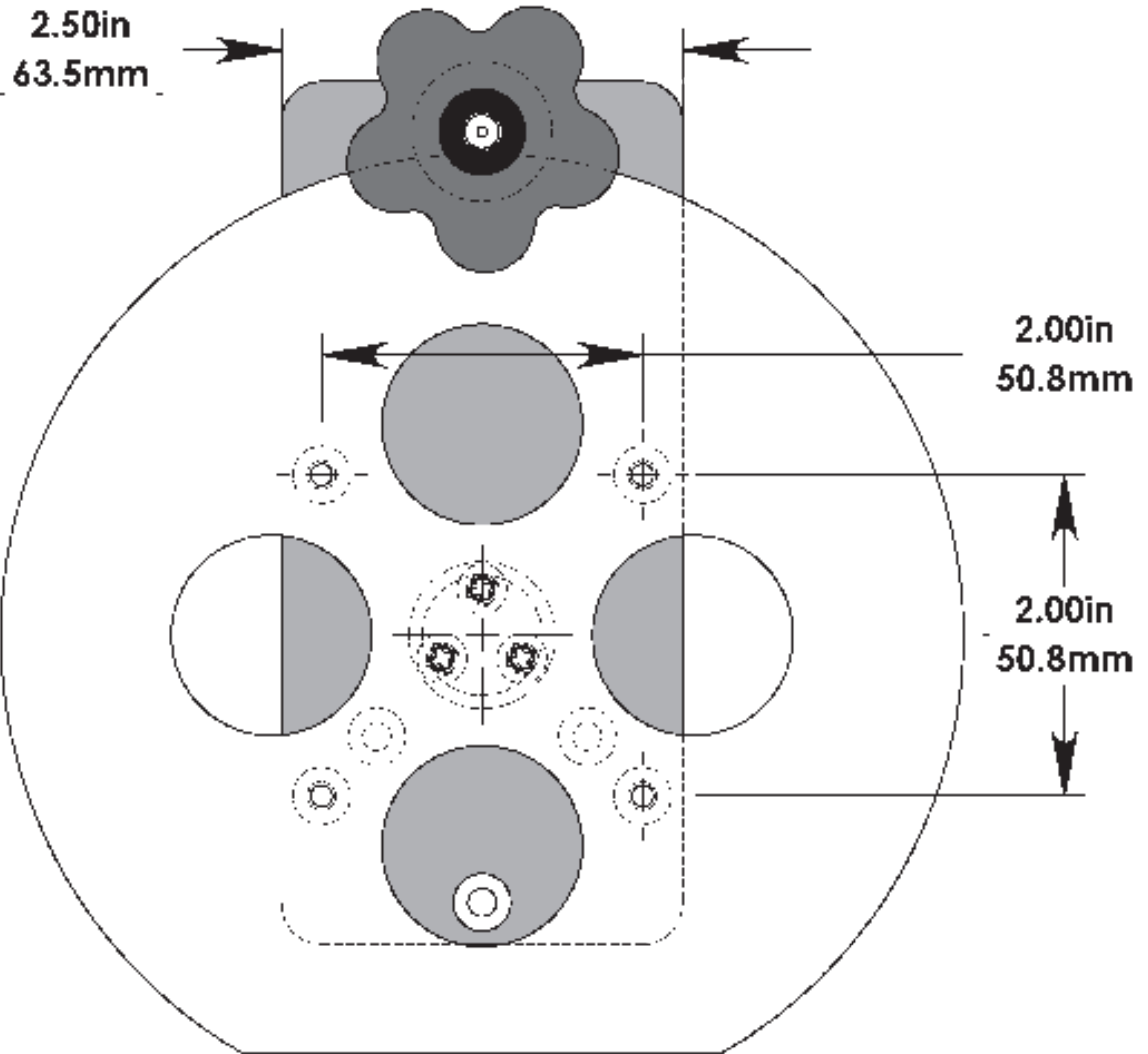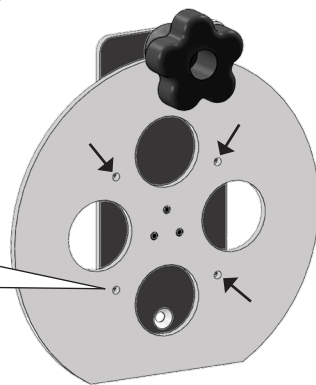

Mount'n Mover™
by BlueSky Designs

mountnmover.com

Product: DP-UR, Universal Rotator
Weight: 7.6 oz. (0.22 kg)

1-888-724-7002 (toll free) or 1-612-724-7002

pg 1 of 2

Universal Rotator (UR)

Attach device plate DP-QCR to device
(shown with Accent 1400)

To adjust rotation:

1. Loosen knob
2. Rotate device
3. Tighten knob

1. Loosen knob
2. Rotate plate to expose device attachment holes

3. Screw UR to device plate (DP-QCR)

8-32 x 1/2 (x2)

ATTACH QRP

Screw QRP to UR.
Four screws included
with QRP (8-32x3/16)

* If you use too long of
screws the rotator will
not turn!

Orient rotator
flat to bottom of
QRP