

Trekker Attachment

Use the seat pan attachment holes. Remove the screws. You will require longer screws; [what is the current screw thread and length?]

Use one or two Adapter Plates (AP, AP2, or AP4) plus an L-Angle Extension Plate to orient the Wheelchair Bracket or attach another plate to move the WB2 so the post is where you want it.

2 options:

1. Attach on top of the seat pan: Remove the screws. Position the plate on top of the seat pan with the threaded holes to the lateral side. Attach with longer screws that match the threaded inserts.

2. Mount below the threaded inserts: Bolts must be long enough to extend beyond the threaded insert enough so a plate can be positioned (through two non-threaded holes) and lock nuts can be attached to the bolts below the plate.



