

Attaches to:

Tobii C8, C12, CEye (eyegaze),
P10, LightWriter SL40, S32

Device (C8,
C12, CEye)
to device
plate



M4-.7 x 25mm (x4)
1/4" x 11/16" spacer, aluminum (x4)

Device (P10,
S32, and
Lightwriter) to
device plate




M4-.7 x 10mm (x4)

Quick release
plate (QRP) to
device plate



8-32 x 1/4" (x4)

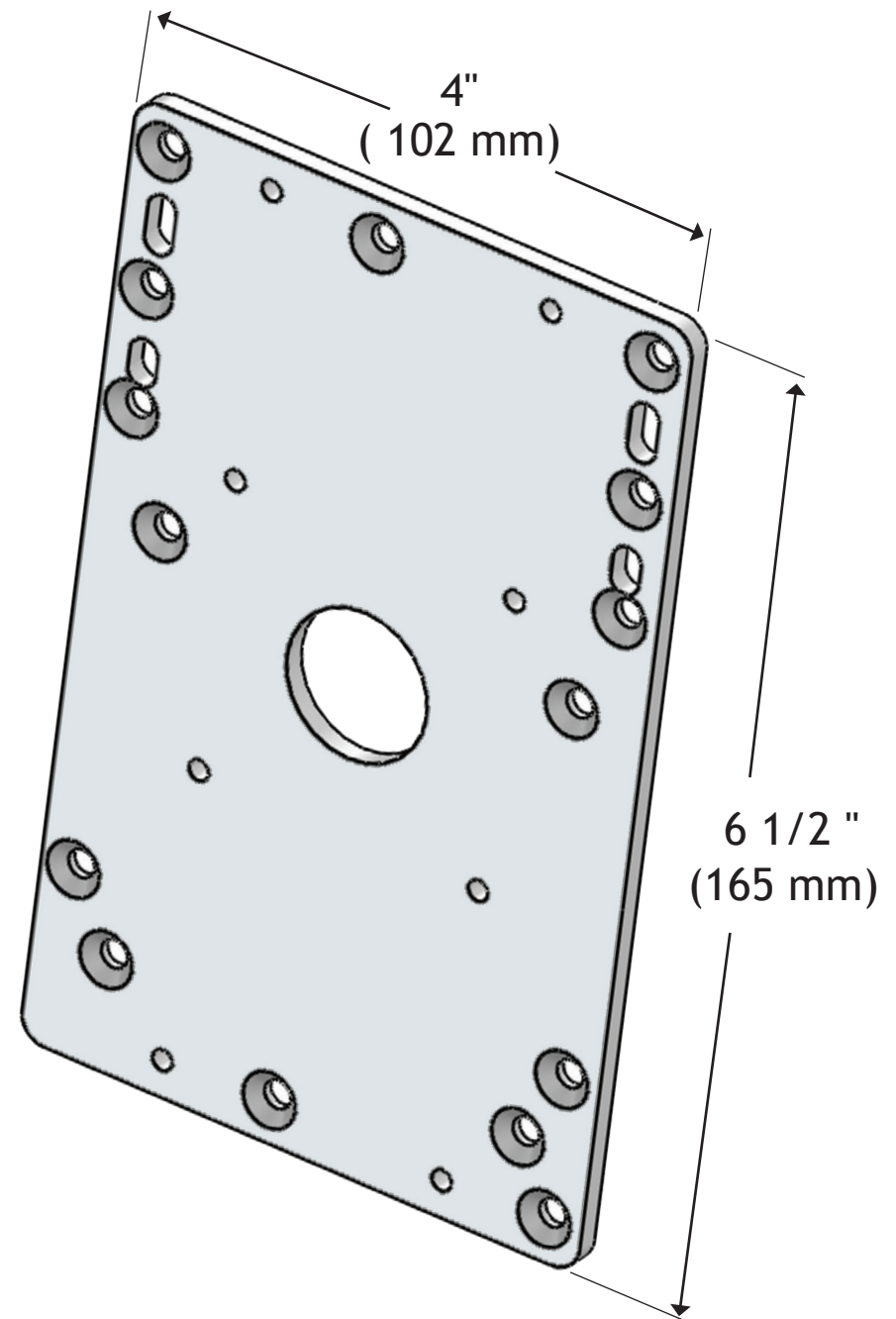

Mount'n Mover™
by BlueSky Designs


mountnmover.com

Product: DP-T1, Tobii
Weight: 5.8 oz. (0.16 kg)

1-888-724-7002 (toll free) or 1-612-724-7002

pg 1 of 5

C8

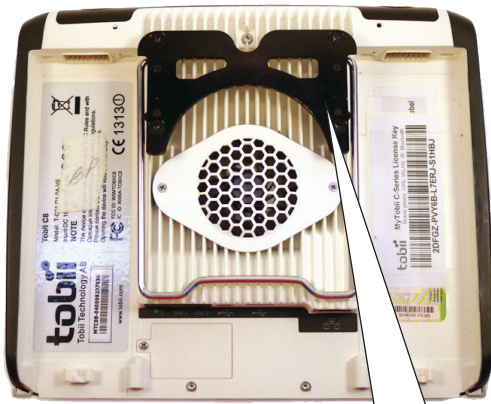

Remove "kick-stand" plate
Save the screws

C12




Leave stand in place
as a spacer on C8 and C12


Plate removed




Attach device plate DP-T1

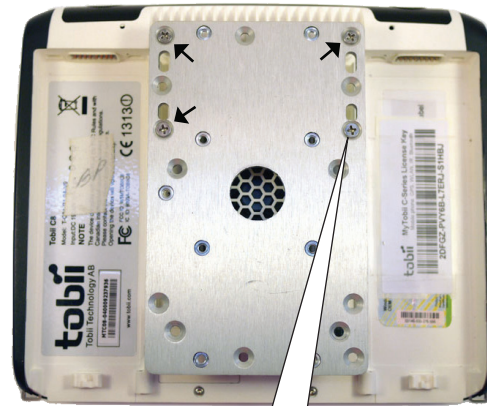

Attach QRP

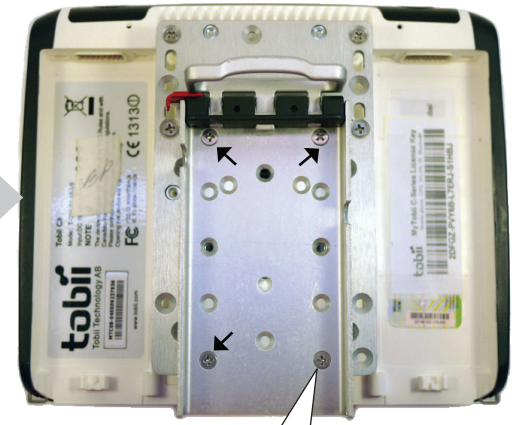


Attach device plate DP-T1 with
existing "kick-stand" plate screws

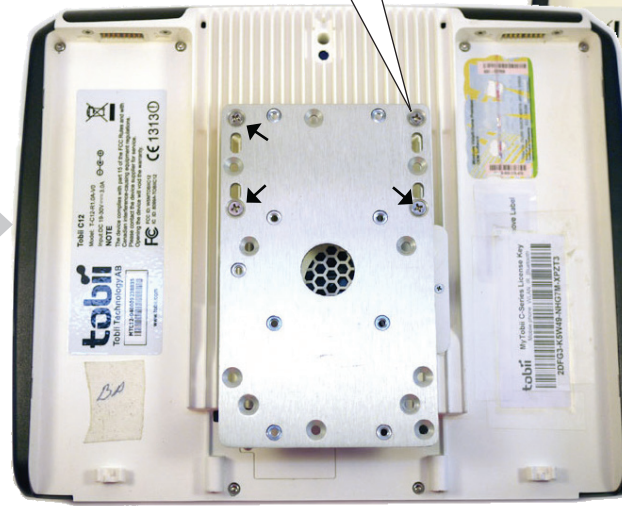

x4
8-32 x 1/4"

Attach QRP

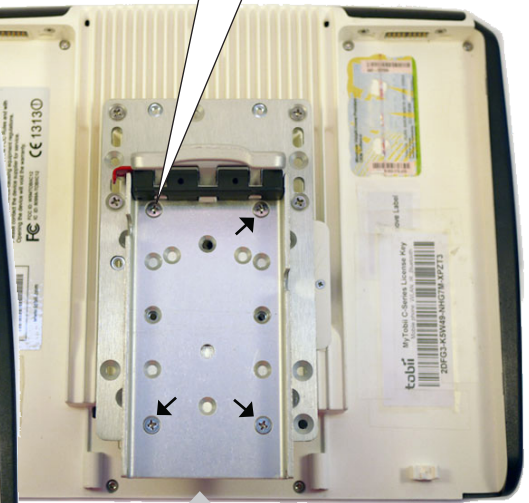

C12 with cEye (eyegaze)

Remove “kick-stand” plate
Save the screws


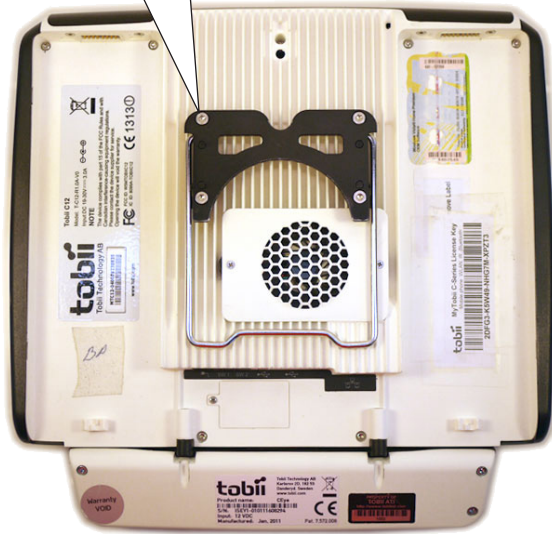


Plate removed



Attach DP-T1

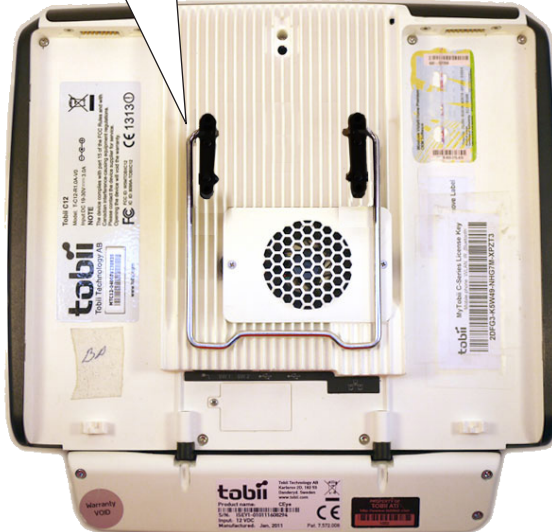


Attach device plate with
existing “kick-stand”
plate screws

x4
8-32 x 1/4”





Attach
QRP

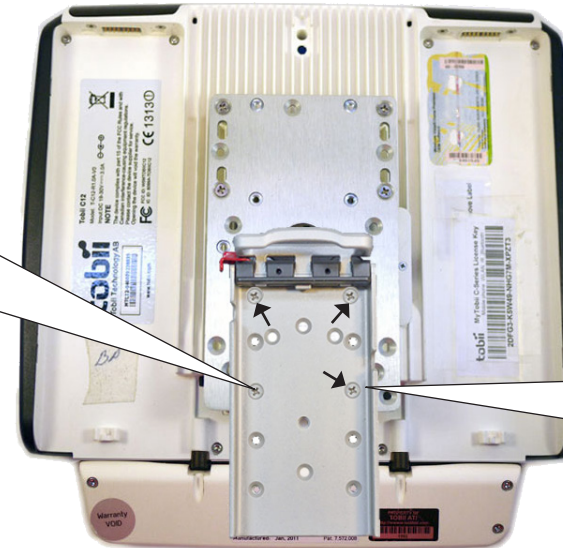

P10

✱ Remove any mounting hardware that may be attached to the device.

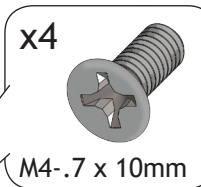
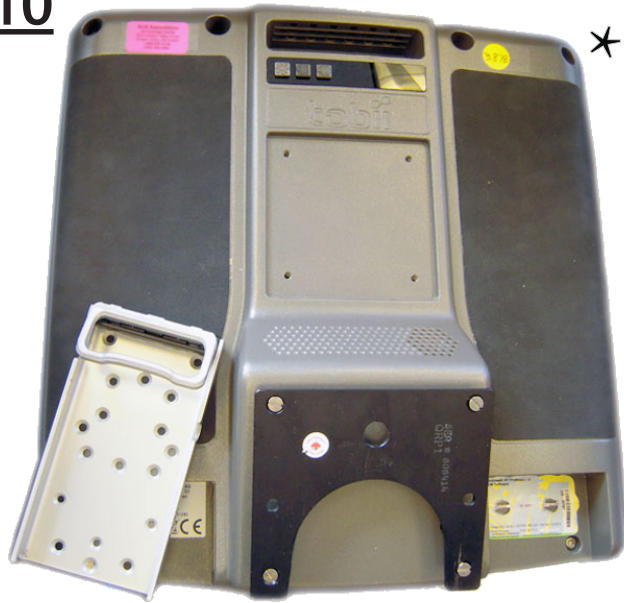


ATTACH DP-T1



ATTACH QRP

LightWriter SL40

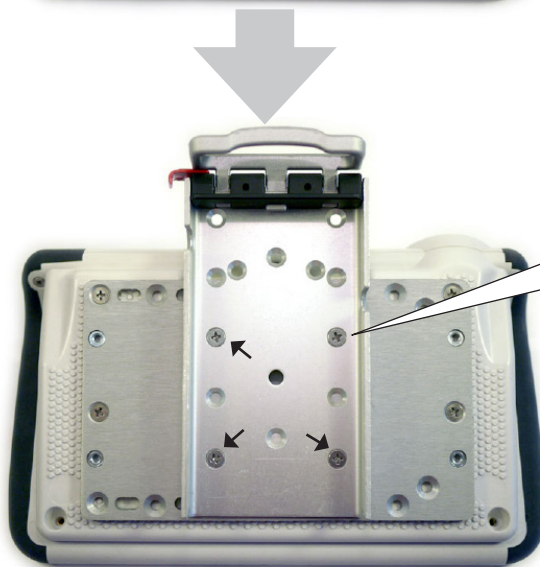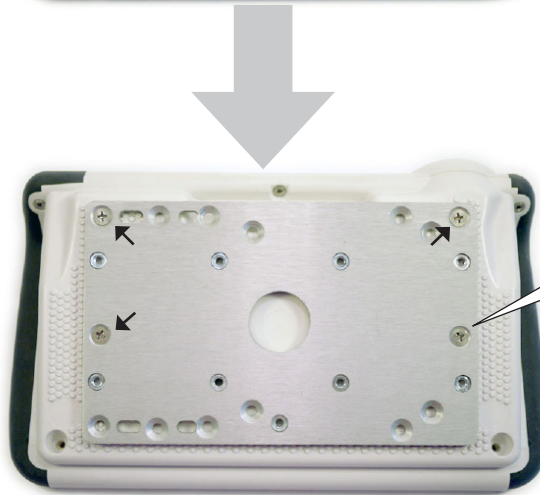

Attach Tobii device plate (DP-T1)

x4
M4-.7 x 10mm

Attach quick release plate (QRP)

x4
8-32 x 1/4"

S32

