




Jabbla Vibe 12 Instructions

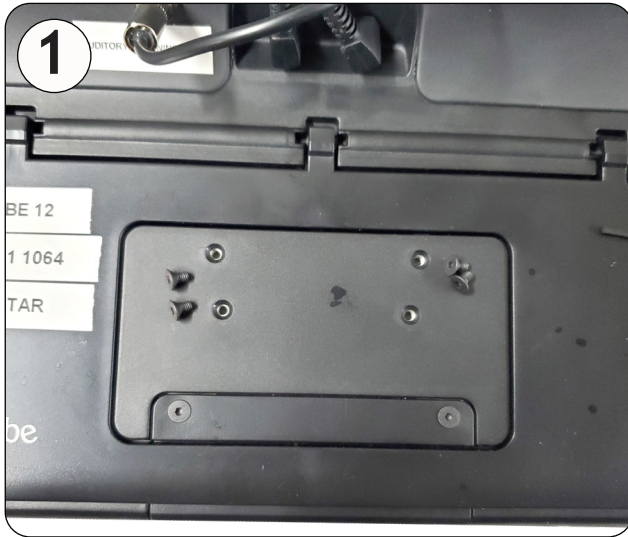

Back of Vibe 12. Remove small screws from plate. Leave plate in place.

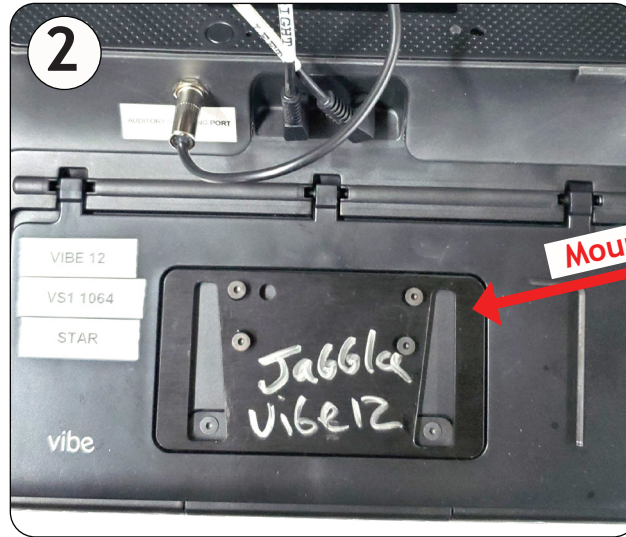


Attach the mounting bracket provided by Jabbla. Use the longer screws that come with the bracket.

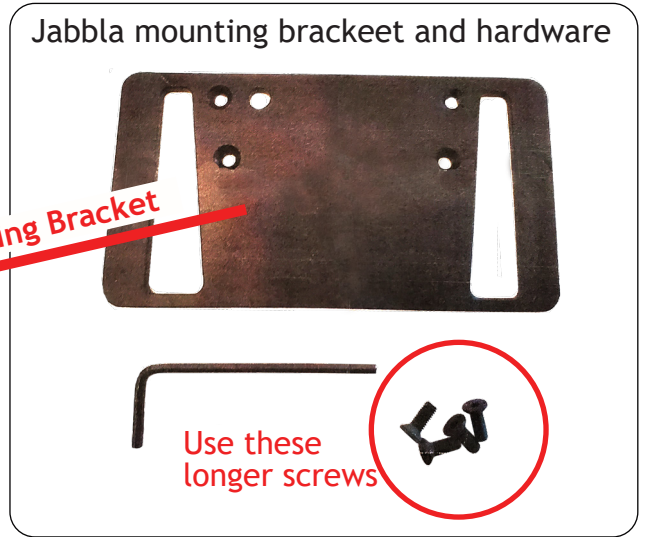

This is the Jabbla "Readapt" mounting bracket. It is required to connect to the Mount'n Mover Quick Connect Receiver.



Attach the DP-QCR (Quick Connect Receiver) to the Tobii mounting bracket. Red pin should "click" into the bracket

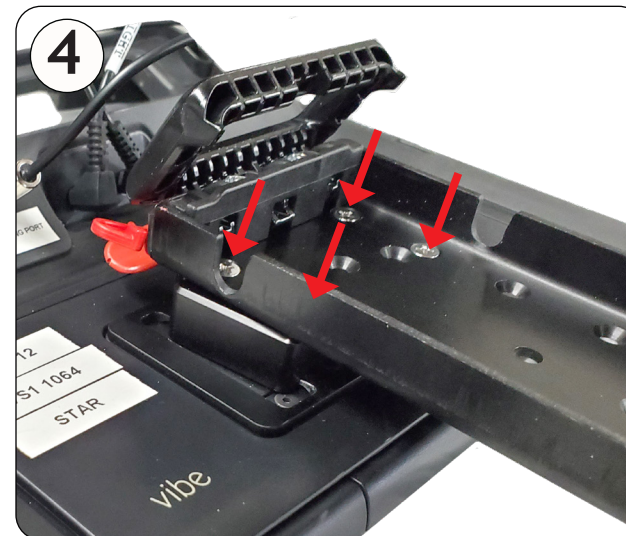

Attach the Quick Release Plate to the DP-QCR using four screws 8-32 x 1/2" included with the DP-QCR



Mount the Quick Release Plate onto the Mount'n Mover tilt plate.