


## About the Laptop Tray (TR-LT)

The Laptop Tray (TR-LT) is designed to attach to a wide range of full sized laptop computers, including tablet and "flip-screen" computers. Keep in mind that a Quick Release Plate (QRP) is required to attach your Laptop Tray to a Mount'n Mover

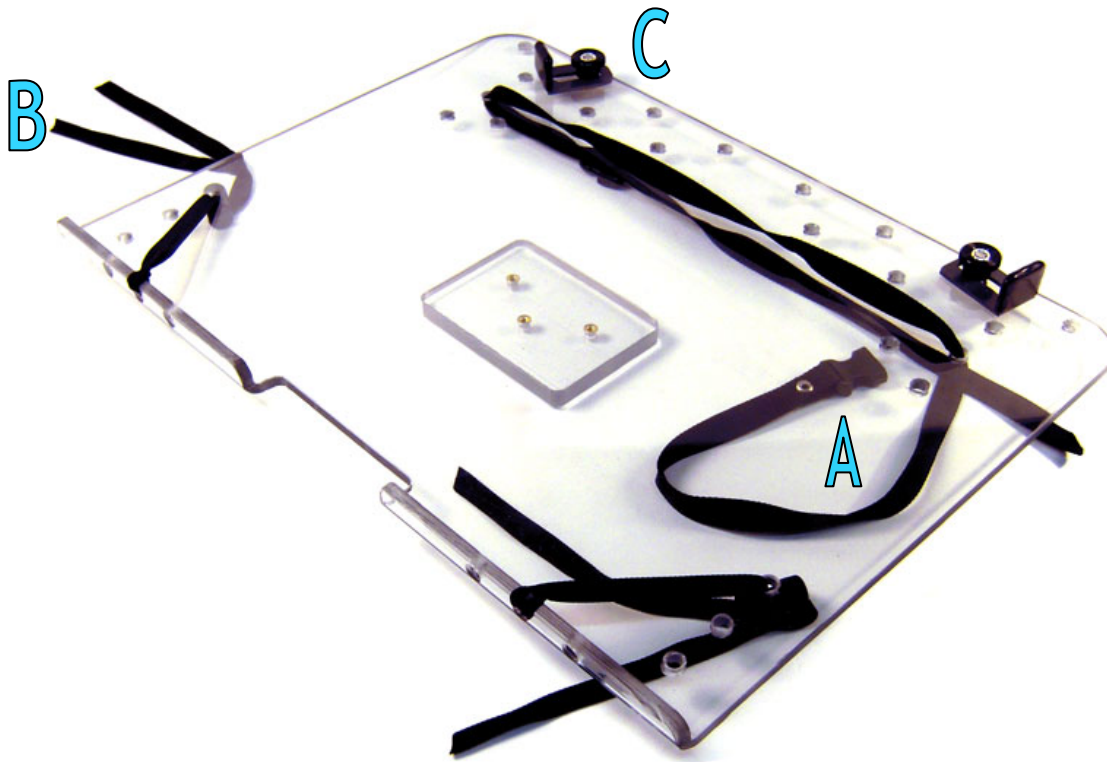

### TR-LT Contents:

1. Laptop Tray
2. Hardware:
  - A. (1x) – **Long strap with buckle**  
*For securing the center of the Laptop*
  - B. (2x) – **Corner straps with barrel-loc**  
*For securing the corners of the Laptop*
  - C. (2x) – **Back-stops**  
*To prohibit the laptop from slipping out.*
  - D. (3x) - **#6-32 x 3/8" Phillips Flat Head Screw**  
*For attaching the Quick Release Plate (QRP) to the TR-LT*

### Requirements:

**Time :** Approx 5 minutes

#### **Other Components:**

- Quick Release Plate (QRP)

#### **Tools :**


- Phillips Screwdriver  
*(#2 is recommended)*



2637 27<sup>th</sup> Ave. S. Suite 209 • Minneapolis, MN 55406  
Ph: 612-724-7002 • Toll Free: 888-724-7002 • Fax: 888-311-4435  
info@blueskydesigns.us • www.mountnmover.com

Rev. 10/13/2010

# Attaching the Laptop to the TR-LT

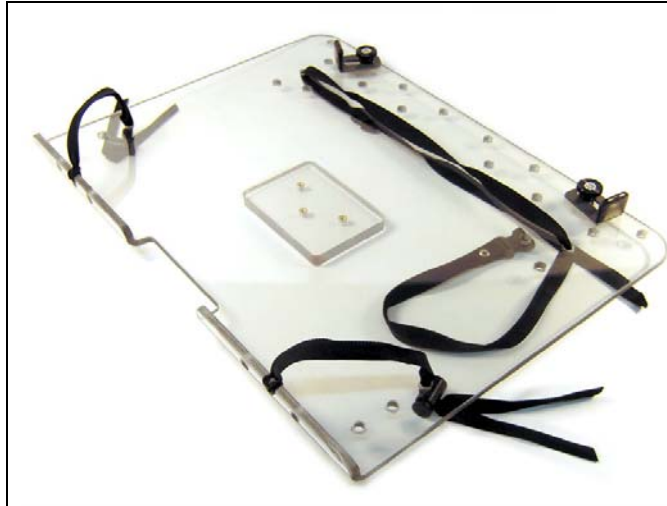

## **Prepare the TR-LT for attachment.**

Loosen the corner straps and unbuckle the long strap with buckle to allow the laptop to be slipped in.

These instructional images do not include a Quick Release Plate (QRP) attached. The same instructions apply if one is attached or not. [QRP to TR-LT instructions can be found here.](#)



## **Step 1/6 :**

With the laptop open, slip over the backstops and under the long strap.



## **Step 2/6 :**

Position the laptop so that it sits against the short laptop tray lip, roughly centered on the tray.



**Step 3/6 :**

Tighten both corner straps using the barrel-loc fasteners.

To use the barrel-loc fasteners, depress the button on it (by pinching it) and then pull the straps to tighten.



**Step 5/6 :**

The image shows the tray flipped upside-down for clarity.

Clasp the center strap using the buckle, and tighten down on the strap.

To tighten the strap, pull the loose end up against the buckle.



**Congratulations!**

Your laptop is now attached to the TR-LT

The Laptop and TR-LT are now ready to be mounted to a Quick Release Plate (**QRP**)

Once the Quick Release Plate is attached to the TR-LT, you are ready to use it on your Mount'n Mover!

If the Laptop Tray is not already attached to a Quick Release Plate, you will need to attach it to one ([instructions can be found here](#)) to use your laptop with the Mount'n Mover.

You are now ready to attach the Quick Release Plate to your Mount'n Mover!  
Refer to page 13 of your Mount'n Mover manual for instructions.  
([you can also view the videos for it here](#))