




Mounting Instructions for R82 Multi Chair

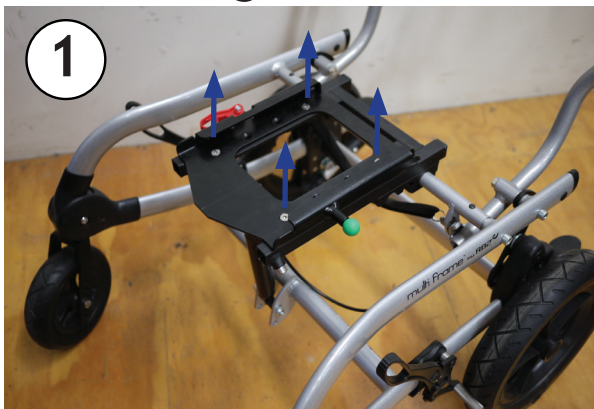


Remove four flat head screws and nuts securing the seat bracket to the base.
Remove bracket from base.

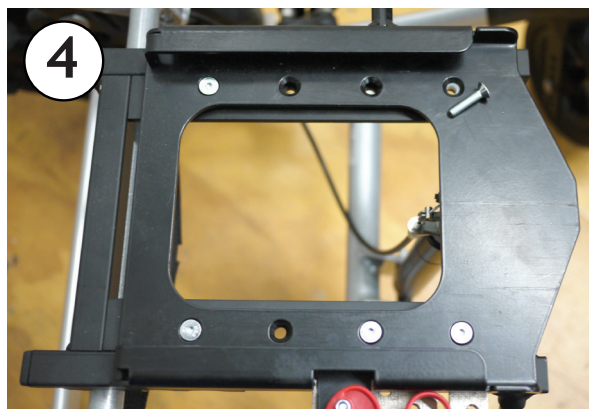


Sandwich AP4 and spacers between the seat and base. Fasten together using five M6 x 25mm flat head screws into M6 lock nuts (provided)

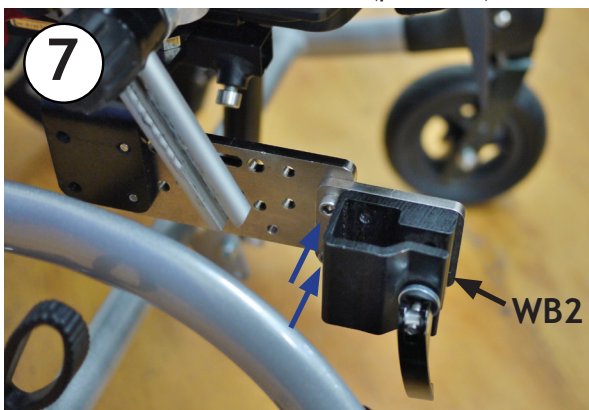

Attach WB2 to the AP2 plate using the provided screws

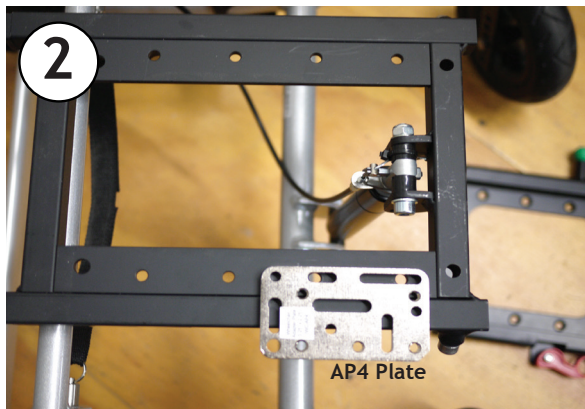


Right-side mount: Place AP4 plate over holes shown above.
(Mirror location for left-side mount)

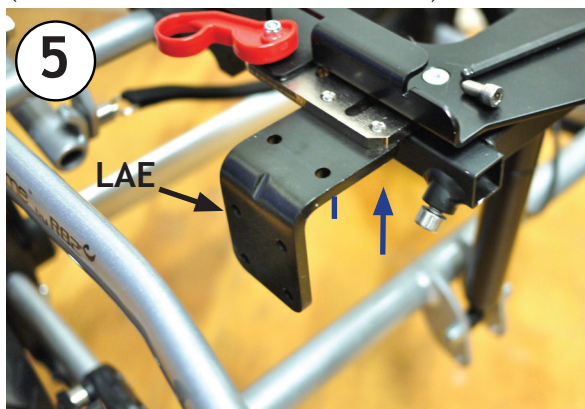


Attach LAE plate to the AP4 plate using provided screws



Insert post into the WB2 and attach the Mount'n Mover to the post!

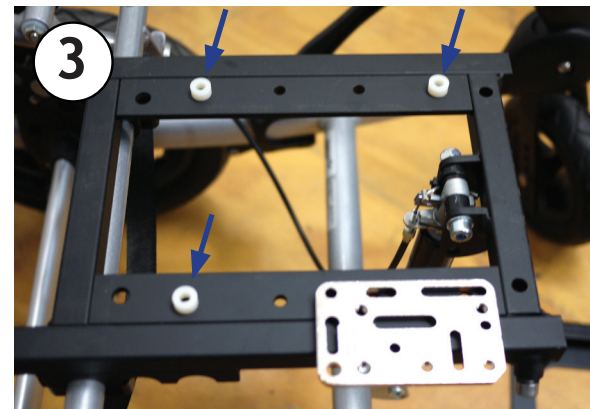

Place 3 nylon 1/4 inch spacers over holes shown above

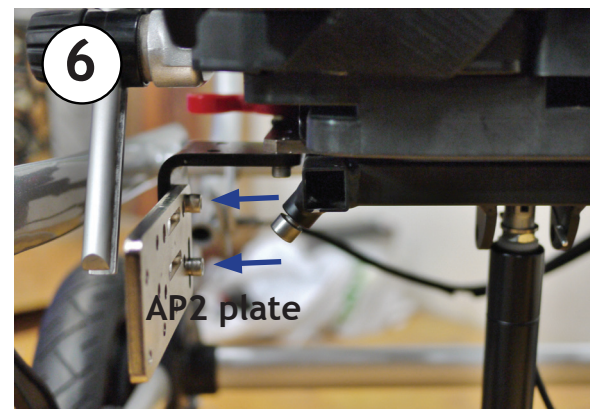

Attach AP2 plate to the LAE plate using the provided screws